Dental Ceramic / Céramique Dentaire

CERABIEN™ MiLai

ENGLISH INSTRUCTIONS FOR USE

I. Introduction

This product is a dental porcelain, which has a potassium-aluminosilicate glass formulation as its primary chemical component. This product is suitable for a wide range of clinical applications, with variable shades. The general clinical benefit of this product is to restore tooth function in the following Indications for Use.

II. Indications for Use

For use in prosthetic dentistry to create an all-ceramic prosthesis.

- Ceramic for coverage of a zirconia substructure
- Ceramic for coverage of a lithium disilicate substructure [CTE: 9.5-11.0x10⁻⁶/K (50-500°C/122-932°F)]

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It is used as a dental porcelain for fabricating dental restorations.

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- Dentists (chairside fabrication of restorations; clinical workflow)
- Dental laboratory technicians (fabrication of restorations in the dental laboratory)

III. Components

For a complete list of the components (including shades and accessories) for this product, see “CERABIEN™ MiLai Baking Schedule”.

IV. Composition

Powder type: Potassium-aluminosilicate glass, Inorganic pigments
Paste type: Potassium-aluminosilicate glass, Aliphatic polyalcohol, Inorganic pigments, Additives
LF LIQUID: Water, oxidizing agents
IS LIQUID: 2-Phenoxyethanol
CERABIEN™ MiLai PASTE LIQUID: Aliphatic polyalcohol

V. Type and Class (ISO6872: 2015)

Type:1 / Class:1

VI. Physical Properties

See Table 2.

VII. Directions for Use

See Table 1 for the Baking schedule.

See Table 3 for instruction of specified liquids.

- Preparation of ceramic framework
 - Condition and adjust the ceramic framework to be used by following the instructions supplied with that product.
- Checking for cracks and cleaning
 - Check the ceramic framework for cracks.
 - After making sure the framework does not have any cracks, sandblast the surface with aluminum oxide (50µm) at a pressure of 0.2MPa (30psi /2 Bar) or less, followed by approximately 10 minutes of ultrasonic cleaning.
 - If any cracks are found, do not use the ceramic framework.

- Wash baking (except for Paint Paste)

Thin, one-layer application of product used as the first layer, and baking is performed.

(For baking described in this instructions for use, follow the baking schedule.)

- Color adjustment

Internal Stain can be used to make color adjustments and characterization, if necessary. Bake it according to Baking Schedule of Internal Stain listed on Table 1.

- Building-up/ Application, Baking

• Build-up or apply to the desired part and bake according to the baking schedule listed on Table 1.

• After baking, the surface of a correctly baked restoration should have a slight luster or sheen.

- Adjusting for morphological corrections, glazing, and completion

• Correct and contour the morphological shape of the restoration, followed by cleaning and glazing. Stains and glazes on surfaces shall be made with Paint Paste.

VIII. Remarks on Handling

Contraindications:

If the patient is hypersensitive to potassium-aluminosilicate glass or any other components, this product must not be used.

Caution and Warning:

- This product should not be used when malocclusion, clenching or bruxism conditions are present.
- If the patient or the dental professional demonstrates a hypersensitivity reaction, such as rash, dermatitis, etc., discontinue use of the product and seek medical attention immediately.
- When mixing materials, or grinding a restoration being fabricated, use an approved dust mask and vacuum with an air filter to protect your lungs from inhaling either dust or liquid.
- When mixing materials or grinding a restoration being fabricated, use safety glasses to prevent dust or liquid from getting into your eyes. If dust or liquid gets into your eyes, immediately rinse with copious amounts of water and seek medical attention.
- Do not use for any purposes except for fabricating dental restorations. This product is for dental application only.
- The use of this product is restricted to dental professionals.
- The restoration is very hot immediately after baking. Do not touch it with your bare hands for a considerable amount of time after removing it from the furnace.
- IS LIQUID is flammable. Do not use near fire or high-temperature objects.
- IS LIQUID is a stimulant liquid. When using it, have good ventilation and if necessary, wear a protective mask, safety glasses and gloves.
- Stains and plaque can accumulate on a restoration after it has been placed in the patient’s oral cavity, depending on the patient’s eating habits or food choices. Instruct patients about the importance of cleaning their restorations.

Precautions in use:

- Do not mix with another dental porcelain in any step of its clinical procedure.
- Porcelain furnace temperature variations may be more significant in the lower ranges. Determine the exact baking schedule by making a preliminary test before baking an actual case restoration.
- The correct baking temperature for this product varies with the size of the restorations to be baked. Adjust the baking temperature of the furnace, referring to the baking schedule.
- Internal Stain and Build/ Paint Paste should be mixed using the specified liquid. (Use of liquids other than those specified may result in a darker restoration color than desired, after baking.)
- This product is a low-temperature baking porcelain. To prevent discoloration of the porcelain, observe the following conditions:
 - If tissue paper fibers get incorporated into the porcelain slurry during water sorption in the condensation procedure, the fibers will not burn off completely. After drying, check to make sure that no residual tissue fibers remain before baking.
 - Always use freshly mixed porcelain.
- The thickness of the layers applied with this product should not be thicker than that of the ceramic framework.
- Keep the interior of the porcelain furnace clean by periodically firing it above 1,000°C (1,832°F) while empty.
- Dispose of this product as a medical waste to prevent infection.
- Do not use Internal Stain as an external stain, Paint Paste and Build Paste as an internal stain.
- Do not use Paint Paste, Build Paste, IS LIQUID and CERABIEN™ MiLai PASTE LIQUID near fire or high-temperature objects.

Storage:

- The product must be stored in a cool and dry place. Keep away from direct sunlight.
- The product should be stored at 1-30°C (33.8-86°F).
- After using Paint Paste and Build Paste, wipe any remaining paste off the rim of opening of the container and tighten the cap securely to prevent the contents from drying out.
- The product must be stored in an appropriate place where only dental personnel have access.
- The product must be used by the expiration date indicated on the package.
- Do not store Paint Paste, Build Paste, IS LIQUID and CERABIEN™ MiLai PASTE LIQUID near fire or high-temperature objects.

[WARRANTY]

Kuraray Noritake Dental Inc. will replace any product that is proven to be defective. Kuraray Noritake Dental Inc. does not accept liability for any loss or damage, direct, consequential or special, arising out of the application or use of or the inability to use these products. Before using, the user shall determine the suitability of the products for the intended use and the user assumes all risk and liability whatsoever in connection therewith.

[NOTE]

- Refills do not include instructions for use (IFU). We provide the IFU in electronic format on the dedicated eIFU website.
- “CERABIEN” is a registered trademark or trademark of NORITAKE CO., LIMITED.

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*1 96kPa = 72cmHg (29 inchesHg)

*2 The lower limit of the baking conditions is established on the assumption that a single crown is baked. When baking multiple restorations, long span restorations or extensive frameworks, adjust the dry-out time and baking temperature based on the upper limit of the baking conditions.

*3 This is baking temperature for cases when Liner, Opacious Body, Body, Enamel, Translucent, Luster, Clear Cervical and Mamelon are applied additionally.

Note: The above program is only a guideline. Baking temperature may be varied with the peculiarities of different furnace.
Porcelain furnace temperature variations may be more significant in the lower ranges. Determine the exact baking schedule by making a preliminary test before baking an actual case restoration.

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Table 3. Specified liquids

Type	Liquid
Internal Stain	IS LIQUID
Porcelain	LF LIQUID or purified water
Build Paste, Paint Paste	CERABIEN™ MiLai PASTE LIQUID (To adjust viscosity)

ENGLISH	FRANÇAIS	ESPAÑOL
Table 3. Specified liquids	Tableau 3. Liquides spécifiés	Tabla 3. Líquidos especificados
Type	Type	Tipo
Liquid	Liquide	Líquido
Porcelain	Céramique	Porcelanas
LF LIQUID or purified water	LF LIQUID ou eau purifiée	LF LIQUID o agua purificada
To adjust viscosity	Pour ajuster la viscosité	Para ajustar la viscosidad

 **Kuraray Noritake Dental Inc.**
300 Higashiyama, Miyoshi-cho, Miyoshi, Aichi 470-0293, Japan
TEL +81-561-32-8953 FAX +81-561-32-8976

US Distributed by **Kuraray America, Inc.**
32 Old Slip, 7th Floor, New York, NY 10005

Caution: Federal (U.S.A.) law restricts this device to sale by or on the order of a dental professional.